

Step 4:

1. Place the Concentrating Devices in the centrifuge buckets at opposite ends. Ensure the concentrating devices

Executive Series Centrifuge

1.5 minutes - 3800 RPM

Platinum Series Centrifuge

PurePRP® SP Spin 1

Platelet

Concentrate Buffvcoat

are balanced (+/- 3g).

2. Process at:

Step 8


PLEASE DISCARD RED VENTED CAP FROM CONCENTRATING DEVICE BEFORE USE. Always swab self-sealing port with sterile alcohol prior to accessing with a sterile syringe.






Using the filter needle, draw up 6mL* of Citrate Anticoagulant into each 60mL syringe.

*Optional Step: If you want to purge the line in Step 3, draw up only 5mL.

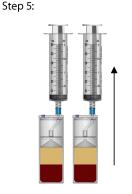


Using the 60mL syringe, aspirate the platelet plasma suspension (PPS) from each device. Aspirate until the RBC fills the aspirating pipe. It is normal to aspirate small amounts of RBC into the syringe during this process


Step 9:

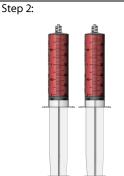



Draw 108mL whole blood from the patient, filling each syringe to 60mL.

Step 6:

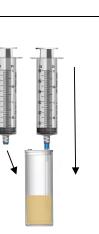

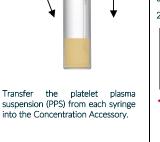

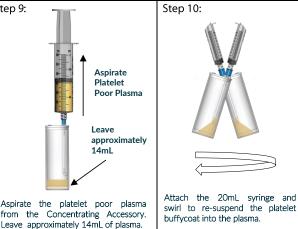

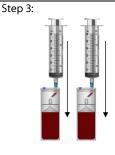

1. Remove the red cap and slowly load the 60mL of anticoagulated whole blood into each Concentrating **Device**

2. Optional Step: Aspirate 2mL Citrate Anticoagulant into 3mL syringe and flush the line from residual blood with Anticoagulant for 1mL each concentrating device (see Step 1).




1. Place the Concentrating Device and Counterbalance Device* in the centrifuge buckets at opposite ends. 2. Process at:

> **Executive Series Centrifuge** 7 minutes - 3800 RPM **Platinum Series Centrifuge**

PurePRP® SP Spin 2 *Counterbalance Device needs to be

filled with water weighing equal to the Concentrating Device (+/- 3g).

Step 11:

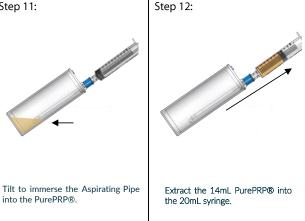

Platelet concentrate buffycoat

separates out at the bottom of the

Concentrating Accessory.

If you have any questions, please reach out to our Clinical Education Team: education@plymouthmedical.com or 888.392.5076 Ext. 3

TEL - 888.392.5076 FAX - 888.392.1455 WEB – PLYMOUTHMEDICAL.COM

PLYMOUTH MEDICAL 135 PLYMOUTH ST, SUITE 410 BROOKLYN, NY11201

into the PurePRP®.

COPYRIGHT © 2020 PLYMOUTH MEDICAL LLC. ALL RIGHTS RESERVED.