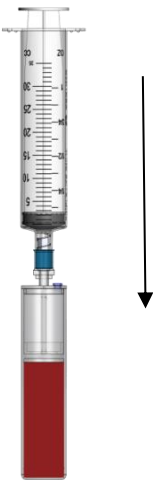
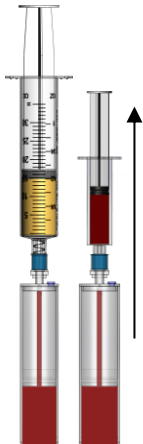
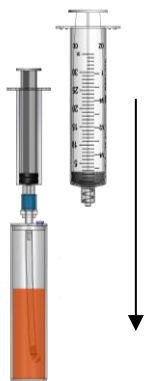
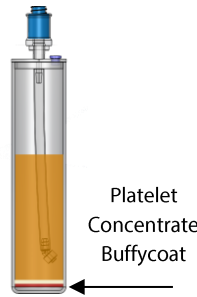
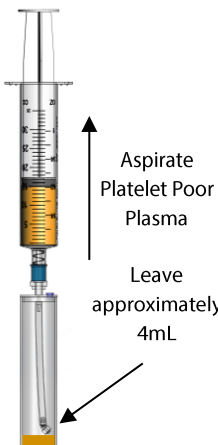
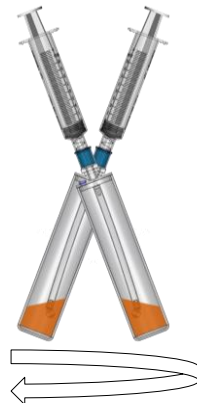
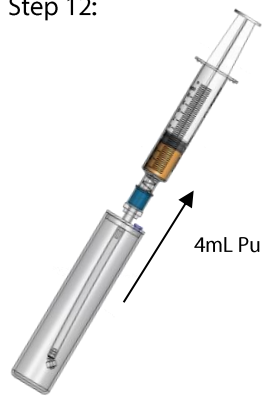


GS30-PURE II PROTOCOL B ** PLEASE DISCARD RED VENTED CAP FROM CONCENTRATING DEVICE BEFORE USE **

Note: Always swab self-sealing port with sterile alcohol prior to accessing with a sterile syringe

Step 1:  Draw 5mL of Citrate Anticoagulant into 30mL Syringe	Step 2:  Draw 25mL whole blood from the patient, filling the syringe to 30mL	Step 3:  Load anticoagulated whole blood into the Concentrating Device	Step 4: Counterbalance and process the Concentrating Device at Executive Series Centrifuge 1 minute 4200 RPM Euro Series Centrifuge 2 minutes 3500 RPM
Step 5:  Using the 30mL syringe, aspirate the platelet plasma suspension (PPS) until RBC fills the aspirating pipe. Then using the 3mL syringe aspirate 0.5mL of RBC	Step 6:  Transfer the platelet plasma suspension (PPS) and the RBC into the Concentrating Accessory	Step 7: Counterbalance and process the Concentrating Accessory at Executive Series Centrifuge 5 minutes 4200 RPM Euro Series Centrifuge 6 minutes 3500 RPM	Step 8:  Platelet Concentrate Buffycoat Platelet concentrate and neutrophil cytokine buffycoat separates out at the bottom of the Concentrating Accessory
Step 9:  Aspirate Platelet Poor Plasma Leave approximately 4mL Aspirate platelet poor plasma from the Concentrating Accessory Leave 4mL of plasma.	Step 10:  Attach the 12mL syringe and swirl to resuspend the platelet buffycoat into the plasma.	Step 11:  Tilt to immerse Aspirating Pipe into PurePRP® Tilt to immerse the Aspirating Pipe into the PurePRP®	Step 12:  4mL PurePRP® Extract the PurePRP® into the 12mL syringe.