



# Coronavirus (COVID-19) Rapid Test

For Medical Professional Use Only

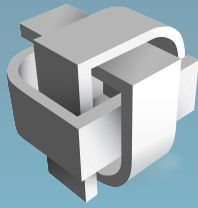

PLYMOUTH  
MEDICAL

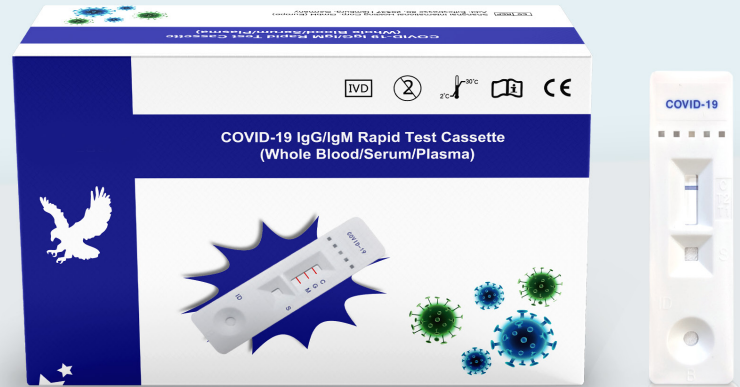

Our easy-to-use COVID-19 test kit provides results in just **10 minutes**



COST-EFFECTIVE



FAST



ACCURATE

## HOW ACCURATE IS THE CORONAVIRUS (COVID-19) RAPID TEST?

Our test has shown in clinical evaluation to have a total agreement of **97.19%** and a kappa value of 0.94.

TEST REAGENT	REFERENCE REAGENT		TOTAL
	POSITIVE	NEGATIVE	
Positive	245	4	249
Negative	16	439	455
Total	261	443	704

Results of the first clinical evaluation, published on March 12, 2020

Clinical evaluation took place across 10 hospitals in China during February 2020 and included 704 tests. Ongoing clinical evaluation and other information is available upon request.

## COVID-19 "CORONAVIRUS" IGG/IGM RAPID TEST KIT

To support public health & safety, we are offering a proven, cost-effective option to detect and help stop the spread of coronavirus.

- Rapid results in 10 minutes
- Following the incubation period, IgM may appear in blood within 3-5 days. IgG will appear as soon as 1-2 weeks.
- Sample: Test can work with whole blood, plasma and serum samples.
- Storage: The kit can be stored at room temperature or refrigerated (2-30°C).
- Shelf Life: 24 months from manufacture date
- Sold in packs of 100
- Forensic/Medical Professional Use Only\*
- EUA approved by the FDA\*\*

## AVAILABLE UPON REQUEST:

- Package Insert
- Clinical Evaluation Report
- Stability Report

## MORE INFO ABOUT OUR RAPID CORONAVIRUS (COVID-19) TEST KIT

For additional information, such as instructions, how-to videos, clinical trial info, the product insert, and CE documentation, ask your sales representative or visit:

<https://www.plymouthmedical.com/covid-19-antibody-test>

\*Forensic/medical professional use only: Tests should be conducted by a licensed phlebotomist, or a medical professional. Verification of use case prior to shipping is mandatory.

\*\*Warning:

- This test has not been FDA cleared or approved;
- This test has been authorized by FDA under an EUA for use by authorized laboratories;
- This test has been authorized only for the presence of IgM and IgG antibodies against SARS-CoV-2, not for any other viruses or pathogens; and
- This test is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Act, 21 U.S.C. § 360bbb-3(b)(1), unless the authorization is terminated or revoked sooner.

More information can be found from the FDA at: <https://www.fda.gov/media/138435/download>

FOR QUESTIONS:

Email: [sales@plymouthmedical.com](mailto:sales@plymouthmedical.com) or Call: 888-392-5076 or Fax: 888-392-1455

PlymouthMedical.com

TO PLACE AN ORDER CLICK [HERE](#)