EmCyte Corporation

PL-42-24

REV.11

Operation Manual

Executive Series Centrifuge II

PN: GS-022624340

5702

5702E

EXSBK-2B

EXSBK-2C

EXSBK-90

EXSBK-90C

Contents

Intended Use	Page 2
Indications	Page 2
General Safety	Page 2
Installation	Page 3
Operation	Page 4
Maintenance	Page 5
Technical	Page 6
Warranty	Page 7
Preventative Maintenance Log	Page 8
Troubleshooting Guide	Page 9

INTENDED USE

The Executive Series Centrifuge II and the Accelerate Executive Series Centrifuge are intended exclusively for indoor use and for separating autologous whole blood and bone marrow aspirate in the EmCyte Component Concentrating System.

The centrifuge may only be operated by trained staff. The operating manual should be read carefully, regardless of whether or not the individual(s) is familiar with the centrifuge.

INDICATIONS

The Executive Series Centrifuge II and the Accelerate Executive Series Centrifuge systems are used in conjunction with the EmCyte Component Concentrating System in the safe and rapid preparation of autologous platelet rich plasma (PRP) from a small sample of blood and/or bone marrow aspirate at the patient's point of care. Autologous PRP, isolated through the use of the Executive Series Centrifuge II and the Accelerate Executive Series Centrifuge systems and the EmCyte Component Concentrating System, can be mixed with autograft and allograft bone prior to application to an orthopedic surgical site or as deemed necessary by the clinical use requirements.

GENERAL SAFETY

To ensure that the centrifuge is operated in a safe manner, avoiding the hazard of injury to personnel or damage to material goods, the following safety precautions should be followed at all times:

- The centrifuge must only be opened by trained personnel competent in its use.
- Never use the centrifuge unless the rotor is properly mounted and secure.
- Never open, or attempt to open the lid while the centrifuge rotor is still turning.
- Only use original parts for the centrifuge.
- Never operate the centrifuge if components are missing or damaged.
- If the rotor becomes damaged it must be discontinued and replacement fitted.
- Only use EmCyte Concentrating Devices. Make sure that concentrating devices are balanced with equal volume and weight prior to centrifugation. Follow product IFU.
- No changes should be made to the mechanical or electrical components.
- The centrifuge has not been manufactured from inert materials or to be explosion proof.
- Ensure that it is not operated within an environment, or with materials that makes these a requirement.
- **ONLY centrifuge whole blood or bone marrow aspirate in the Executive Series Centrifuge II and the Accelerate Executive Series Centrifuge.**
- Do not centrifuge corrosive, radioactive, flammable, or explosive substances. Do not centrifuge substances prone to react briskly with each other. Should any doubt exist regarding the suitability of a substance, consult safety personnel or call customer support to comply with national regulations (refer to World Health Organization: Laboratory Biosafety Manual).
- Concentrating device and/or Counter Balance **MUST** be loaded into the centrifuge **OPPOSITE** of each other and with each one containing the same volume. Failing to do so may result in damage to the centrifuge and/or personnel injury.
- In order to prevent damage to the compressor following improper transport, the device may not be switched on until four hours after setting up.
- Please remove the transport safety device and keep it for possible use if the device is subsequently moved again.
- Place the centrifuge on a solid, flat, non-resonant lab bench. The condensation water tray should be inserted from the side.
- The surrounding area must be well ventilated and protected against direct sunlight.
- To ensure that the ventilation of the device is not impaired, a fundamental clearance of 30 cm to the side and a minimum of 15 cm to the back wall must be maintained.
- During centrifugation, according to the recommendations set out in EN 61010-2-020 a safety clearance of 30 cm must be maintained around the centrifuge within which there are no

- objects which may be destroyed and so cause further damage.
- Only connect the device to power supplies which correspond with the electrical requirements on the name plate.
- Only use sockets with a protective earth (PE) conductor and suitable power cable.

DISPOSAL OF CENTRIFUGE

- **Domestic**
In the event that the centrifuge needs to be disposed of, contact the local state's disposal agency and follow the state's local disposal regulations of properly disposing an electronic medical equipment. PLEASE DO NOT DISPOSE THE CENTRIFUGE IN REGULAR TRASH.
- **International**
In accordance with EU Directive 2002/96/EC on waste electrical and electronic equipment (WEEE), this product cannot be disposed of in municipal or domestic waste. They are marked with the following symbol to indicate this:

As disposal regulations within the EU may vary from country to country, please contact your supplier if necessary.

AEROSOL CAPS

- Aerosol caps are used to prevent biologic aerosol inside the centrifuge machine container. Check the aerosol caps regularly for damage. Damaged aerosol caps must be replaced immediately. It is recommended that Aerosol Caps be used during the processing of blood components.

INSTALLATION

The instrument has been designed in such a way that installation should present no problems to a technician familiar with equipment of this type. Should any problems or questions arise during the installation of the instrument please call EmCyte Corporation's customer support.

WORK SURFACE

The work surface must be stable and, where possible, resonance free. A standard laboratory bench is usually suitable. The location of the centrifuge should be well ventilated and out of direct sunlight, having sufficient room to allow air to circulate around it freely. A safety zone extending for 30cm in all directions must be maintained around the centrifuge. This space should be kept clear; in particular no hazardous material may be placed in the area during centrifugation.

MAIN ELECTRICAL CONNECTION

Before connecting the unit to the main electrical connection, ensure that the voltage and frequency of supply match the specifications on the instrument.

ROTOR

The rotors and buckets must be loaded symmetrically. The adapters must be loaded only with the specified EmCyte Concentrating Devices. Symmetrical Concentrating Devices must have equal weight. This will reduce wear on the drive and eliminate running noise. The centrifuges have an automatic imbalance detector, which shuts down the centrifuge if the weight differences are excessive. This is indicated by the error message in the display. Rotors are high-grade components which have to withstand extreme stresses and strains. Aluminum rotors are largely protected from corrosion by the most common laboratory chemicals by means of an anodized coating. This protection is not unlimited. Protect the rotors from mechanical damage. Even minor scratches or cracks can result in serious internal material damage. Avoid damaging the rotors by the use of aggressive chemicals, such as: strong and weak alkalis, strong acids, solutions of mercury, copper and other heavy metal ions, chlorinated hydrocarbons, concentrated salt solutions and phenol. If the rotor is contaminated by aggressive substances, clean it immediately with a mild cleaning agent.

SWING BUCKET ROTOR

The swing-bucket rotor is equipped with round or rectangular buckets. Make sure that the buckets are properly installed, lubricated lightly and can swing out freely. Carry out a manual swing test with empty Concentrating Devices. Avoid unbalanced loading of the Concentrating Devices. Before inserting the buckets in the grooves, make sure they are clean. Dirty grooves will prevent the buckets from swinging out evenly.

OPERATION

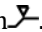
CENTRIFUGATION WITH TIMER SETTING

Switch on the centrifuge, with the mains power switch if necessary, and then with the switch. The nominal values of the last run are displayed. Load the rotor symmetrically and close down the centrifuge lid.

1. Time - Alters the running time.
2. Speed - Alters the speed in increments of 100 1/min or RCF.
3. Start - Starts the run. The symbol flashes while the rotor is running.
4. Stop - Stops the centrifuge. The symbol appears briefly as soon as the rotor comes to a standstill.
5. Stand-by - Centrifuge switched to stand-by mode.
6. Open - Releases the lid latch.

While the centrifuge is running the remaining time is displayed. The last minute is counted down in seconds. When the preset time elapses the centrifuge stops automatically and emits a signal tone to indicate the centrifugation is finished. The lid opens automatically following centrifugation. When you switch off the centrifuge at the main power switch the display goes blank after a few seconds' delay.

CENTRIFUGATION WITH "AT SET RPM" SETTING

For centrifugation, time can be counted either immediately from the start or not until attainment of the preset speed. Pressing the START / STOP button for longer than 2 seconds with the centrifuge lid open switches the "at set rpm" mode, symbolized by the pictogram .

To exit the "at set rpm" mode again and begin counting down the centrifugation time immediately after starting the centrifuge, press the START / STOP button again for longer than 2 seconds with the centrifuge lid open until the pictogram  is displayed.

EMERGENCY LID RELEASE IN CASE OF LID LOCK FAILURE

The lid can be manually opened in the unlikely event of the lid lock failing and the user being unable to remove their samples. Emergency lid release on underside of device by pull cord: In front of the front right suction foot, there is a small white plastic cap in the base plate. Remove the cap and draw the cord out straight downward. Press the plastic button into the base plate again.

Note: Please ensure that the cord is pushed back completely into the housing before closing the lid.

WARNING: Operation of the emergency lid release must only be undertaken by qualified and approved personnel. Under no circumstances may the centrifuge be operated in the condition. Wait until the rotor has stopped turning (view through the lid port) and switch off the power to the centrifuge. Pull the release cord to open the lid.

WARNING: Under no circumstances must the emergency lid release be used while the centrifuge is running.

MAINTENANCE

CLEANING, DISINFECTING AND MAINTENANCE

It is also recommended that the rotor chamber be cleaned after each point of care use. The end user is responsible for appropriate cleaning and decontamination. Before cleaning, unplug the power plug with the lid open, unscrew the rotor using the rotor key supplied and clean it separately. Use a mild cleaning agent for cleaning the accessible surfaces of the device, for example, alcohol (ethanol, isopropanol) or alcohol-based disinfectants. Do not allow any liquid to get into the gap at the motor shaft outlet. For this reason, the rotor chamber should be cleaned only with a damp cloth. In order to ensure long-term, reliable work with your centrifuge, please note that aggressive chemicals may damage the rotor and the chamber. Check your device once a month for corrosion and damage. The rubber seals in the rotor chamber should be rinsed off thoroughly with water and lubricated with glycerin or talc after every clean to prevent them becoming brittle.

The rotors, buckets, tube holders and adapters should be cleaned after every use or when necessary with a neutral cleaning agent to prevent residues of the material being centrifuged from changing the properties of the centrifuge and its accessories. The rotor must be taken off for this. Never place components in disinfectants or cleaning agents which contain sodium hypochlorite/chlorine or are oxidants. This will cause the material to change. Disinfection with glutaraldehyde solution is possible. We recommend Cidex® activated glutaraldehyde solution. The plastic adapters and rubber plates are dishwasher safe. Check the tube holders and buckets for residues and corrosion. For thorough cleaning, clean all parts separately. Then refit the rotor and bolt it into place with the supplied rotor key.

WARNING: ISOLATE AND DISCONNECT THE POWER SUPPLY BEFORE CARRYING OUT CLEANING OR MAINTENANCE.

CENTRIFUGE BODY

- Regularly wipe down the external surfaces with a mild detergent.
- Mop up any spillage with an absorbent cloth. Check to ensure that the liquid has not entered the main body of the centrifuge.

WARNING: If any liquid has entered the main body of the centrifuge it **MUST NOT** be used. Contact the supplier or EmCyte Corporation immediately for advice.

ROTOR

- Clean the rotor using a suitable detergent and hot water. Alternatively the rotor may be cleaned in an autoclave at a maximum temperature of 121 C° for 20 minutes.
- Thoroughly rinse and dry the rotor.
- Inspect the rotor for any hairline cracks, damage or other defects. If any are found please contact your supplier or EmCyte Corporation Inc.

TECHNICAL**TECHNICAL SPECIFICATIONS**

Power output:	200W
Max. Speed:	4400 RPM
Max. Centrifugal acceleration:	3000 RCF
Max. Load	4 x 90mL
Max kinetic energy:	2 280 Nm
Permissible density of material being centrifuged:	1, 2 g/ml
Ambient temperature:	2 - 40 °C
Maximum humidity:	75%
Dimensions:	Height: 241 mm Depth: 395 mm Width: 320 mm
Weight excluding rotor:	20kg
Startup time (120 V, 230V):	<25 sec
Deceleration time (120 V, 230V):	<25 sec
Noise level:	< 58 dB (A)

WARRANTY

The centrifuge machine comes with a one (1) year manufacturer warranty from the date of purchase by the first consumer purchaser of the product. The warranty only covers the cost of labor and replacement of mechanical or electrical parts that fail during normal operation. The manufacturer may use functionally equivalent reconditioned/refurbished/pre-owned or new products. Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with liquid, water, rain, extreme humidity or heavy perspiration, sand, dirt or the like, extreme heat, or food; (c) use of the products or accessories for commercial purposes or subjecting the product or accessory to abnormal usage or conditions; or (d) other acts which are not the fault of the manufacturer, are excluded from coverage. Defects or damages resulting from service, testing, adjustment, installation, maintenance, alteration, including without limitation, software changes, or modification in any way by someone other than the manufacturer, or its authorized service centers, are excluded from coverage. The service is done at the manufacturer's repair facility, not on site. The customer pays for shipping and handling.

PREVENTATIVE MAINTENANCE & CARE

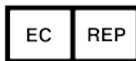
The preventative maintenance & care is done on site by the operator. A preventative maintenance log must be kept at the facility of operation. Provided with the machine is a preventative maintenance & care instruction sheet. Each item must be performed and recorded. Preventative maintenance services are done according to facility schedule.

LIMITATIONS

Any implied warranties, including without limitation the implied warranties of merchantability and fitness for a particular purpose, shall be limited to the duration of this limited warranty, otherwise the repair or replacement, as provided under this express limited warranty is the exclusive remedy of the consumer, and is provided in lieu of all other warranties, express or implied. In no event shall the manufacturer be liable, whether in contract or tort (including negligence) for damages in excess of the purchase price of the product, accessory or software, or for any indirect, incidental, special or consequential damages of any kind, or loss of revenue or profits, loss of business, loss of information or data, software or applications or other financial loss arising out of or in connection with the ability or inability to use the products, accessories or software to the full extent these damages may be disclaimed by law.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician. The physician is solely responsible for the use of this device

This system contains: 1 Executive Series Centrifuge, 1 Centrifuge rotor, 4 hanging buckets with caps.

Authorized Representative
Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Eppendorf Manufacturing Corp.
175 Freshwater Boulevard
Enfield, CT 06082, USA
Phone: 860-253-3400

Australian Sponsor
Emergo Australia
Level 20
Tower II, Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia

EmCyte®

Executive Series Centrifuge II PREVENTATIVE MAINTENANCE LOG

The outer surfaces of the centrifuge and the rotor chamber should be cleaned regularly with a mild cleaning agent. This is for hygiene purposes and to prevent adhering impurities causing corrosion. If material hazardous to health or aggressive material contaminate the device, the owner is responsible for appropriate cleaning and decontamination. Before cleaning, unplug the power plug with the lid open, unscrew the rotor using the rotor key supplied and clean it separately. Use a mild cleaning agent for cleaning the accessible surfaces of the device. Choose disinfection methods that correspond to the current legal regulations and guidelines for your area of application. For example, use alcohol (ethanol, isopropanol) or alcohol-based disinfectants. Do not allow any liquid to get into the gap at the motor shaft outlet. For this reason, the rotor chamber should be cleaned only with a damp cloth. Never place components in disinfectants or cleaning agents which contain sodium hypochlorite/chlorine or are oxidants. This will cause the material to change. Use protective equipment, including personal protective equipment for eyes, face, head, and extremities, and protective shields and barriers when handling chemical hazards or blood contaminants.

MAINTENANCE DESCRIPTION	MAINTENANCE COMPLETED	DATE COMPLETED	SIGNATURE
Unplug power plug with lid open	Y <input type="checkbox"/> N <input type="checkbox"/>		
Protective equipment has been employed	Y <input type="checkbox"/> N <input type="checkbox"/>		
Mild cleaning agent has been attained (i.e. alcohol or alcohol based disinfectants)	Y <input type="checkbox"/> N <input type="checkbox"/>		
Device checked for corrosion and damage	Y <input type="checkbox"/> N <input type="checkbox"/>		
Rotors removed and cleaned	Y <input type="checkbox"/> N <input type="checkbox"/>		
Buckets cleaned	Y <input type="checkbox"/> N <input type="checkbox"/>		
Tube holders cleaned	Y <input type="checkbox"/> N <input type="checkbox"/>		
Adapters cleaned	Y <input type="checkbox"/> N <input type="checkbox"/>		
External surfaces wiped down	Y <input type="checkbox"/> N <input type="checkbox"/>		
Rotor, buckets, caps, seals, and external surfaces inspected for hairline cracks, damage or defects	Y <input type="checkbox"/> N <input type="checkbox"/>		

TROUBLESHOOTING GUIDE

TROUBLESHOOTING GUIDE				
Error	Causes	Solution 1	Solution 2	Solution 3
No Display	No Power	Plug the power cable on the back of the centrifuge & plug in power cable to an outlet.	On the back of the centrifuge, ENSURE that the power switch is turned ON.	On the front of the centrifuge, ENSURE that the power button is lit (it will be green).
Lid Will Not Close	The lid is not completely closed	Open the lid & close it again. Push down slightly on the lid to ensure it is closed.	-----	-----
Stand-by button lit red	Centrifuge is not ready	Press STANDBY button.	-----	-----
Centrifuge shakes	Rotor is unevenly loaded	Stop centrifuge & ensure it is loaded evenly (each device must be opposite of each other).	-----	-----
Centrifuge is very loud	Rotor is loose	Tighten rotor (use the rotor tool and turn it clockwise).	Verify that the blue buckets are placed properly on the rotor.	Verify counter balance weight.
No Blood Separation	On the display, the triangle is NOT pointing down as such 	With the centrifuge lid open, press and hold the power button for 2 seconds	Counter check if the Power is OFF & the Brake is OFF by pressing the SHORT & OPEN buttons at the same time.	-----
Lid will not open	Power failure Rotor still running	Emergency lid release. Wait for rotor to stop.	-----	-----

TROUBLESHOOTING GUIDE CONTINUED

Error	Causes	Solution 1	Solution 2	Solution 3
DISPLAY ERROR MESSAGE:				
LID	<ul style="list-style-type: none"> Lid not released Lid not locked 	<ul style="list-style-type: none"> Press lid down tight, press Start/Stop or emergency release (Pull String release is located on the front right bottom of the centrifuge). Close lid again. 	-----	-----
Er 2	Unbalanced rotor switch	Service is needed.	-----	-----
Er 3	Improper Balance/Error in speed system	Make sure to <u>remove red cap</u> & discard prior to placing concentrating device into the centrifuge.	Turn off front display by pressing power button on the front. Turn off machine by pressing power button in the back.	Wait for the rotor to come to a stop and unplug the power cord and leave it off upto 5 minutes). Plug in the power cord and turn on the power button.
Er 5	Lid Switch	Close lid, restart.	-----	-----
Inb Rotor	Rotor is unevenly loaded	Stop centrifuge & ensure it is loaded evenly.	-----	-----
Int	Power failure during run	Check power plug. Wait for the rotor to come to a stop. Repeat run.	-----	-----
Er 6	Drive error	Repeat run.	-----	-----
Er 7	Overspeed	Error in drive or speed measurement.	-----	-----
Er 8	Drive error	Repeat run.	-----	-----
Er 9-25	Electronics error	Repeat run.	-----	-----
Er 23	Motor overheating	Make sure that the air circulation through the cooling slots is not impaired.	-----	-----

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TECHNIQUE

TECHNICAL SPECIFICATIONS

Power output:	200W
Max. Speed:	4400 RPM
Max. Centrifugal acceleration:	3000 RCF
Max. Load	4 x 90mL
Max kinetic energy:	2 280 Nm
Permissible density of material being centrifuged:	1, 2 g/ml
Ambient temperature:	2 - 40 °C
Maximum humidity:	75%
Dimensions:	Height: 241 mm
	Depth: 395 mm
	Width: 320 mm
Weight excluding rotor:	20kg
Startup time (120 V, 230V):	<25 sec
Deceleration time (120 V, 230V):	<25 sec
Noise level:	< 58 dB (A)

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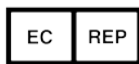
Authorized Representative
Emergo Europe
Prinsesgracht 20
2514 AP The Hague
The Netherlands

Eppendorf Manufacturing Corp.
175 Freshwater Boulevard
Enfield, CT 06082, USA
Phone: 860-253-3400

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